



Conformer
by Prof. Neubauer

Conformer are used for Orbita reconstruction. Shape and size must be decided individually by the doctor.

Available sizes

Size 01 28 mm x 25 mm
Size 02 27 mm x 24 mm
Size 03 26 mm x 22 mm
Size 04 24 mm x 21 mm



**Individual
adjustment**

All tools can be produced in other sizes, shapes and colors on request.

If required, there are also modifications or completely new developments possible.

Instructions for use of the products of PMMA (Polymethylmethacrylate)

Our products are made of heat-sensitive material. Therefore, this material may not be heated above 60° C.

Sterilization with ethylene oxide or gamma rays is not recommended.

Disinfection

Method 1

Our products should be cleaned in an ultrasonic bath with commercially available disinfectants. Then rinse with deionized water or saline solution.

Method 2

10 minute bath in a 3% hydrogen peroxide solution. Then rinse with deionized water or saline solution.

Sterilization

We recommend a low-temperature plasma sterilization (Sterrad® process).
Formaldehyde sterilization only up to a temperature of less than 60° C.

**Other Sterilizations are
not recommended**

Ordering and contact information

Institut für Kunstaugen und
ophthalmologische Hilfsmittel
Nicolaus Müller-Uri

Hirtenstraße 9
65193 Wiesbaden
Germany

+49(0)611-58 08 84 07

+49(0)611-58 08 94 82

@ order@kunstauge-wiesbaden.de

www.kunstauge-wiesbaden.de

www.kunstauge-wiesbaden.de



Institut für Kunstaugen und
ophthalmologische Hilfsmittel
Nicolaus Müller-Uri

Acrylic clinical and surgical products



Dear customer,

our institute manufactures and delivers tools for diagnosis and therapy. Many of these tools created at the suggestion and in cooperation with specialists, under which they were named. Initially made of glass, those tools have been made from medical plastic (PMMA) for more than 50 years.

As in the past, we can always make individual modifications or create completely new products to your specifications and needs.

Please contact us for more details or use this brochure for more information about mostly ordered tools and sizes.




Covering shell

The shells are used during surgery on the eyelids to prevent injury of the eye. Shape and size must be decided individually by the doctor.

Available sizes

- Ø 22 mm
- Ø 25 mm x 23 mm
- Ø 23 mm x 21 mm

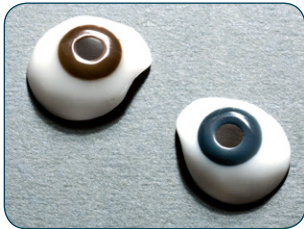

Interim prostheses

Interim Prosthetic Shell

Available sizes

- | | | | |
|---------|---------------|---------|---------------|
| Size 01 | 26 mm x 24 mm | Size 07 | 24 mm x 20 mm |
| Size 02 | 26 mm x 23 mm | Size 08 | 23 mm x 21 mm |
| Size 03 | 25 mm x 23 mm | Size 09 | 23 mm x 20 mm |
| Size 04 | 25 mm x 22 mm | Size 10 | 23 mm x 19 mm |
| Size 05 | 24 mm x 22 mm | Size 11 | 22 mm x 19 mm |
| Size 06 | 24 mm x 21 mm | Size 12 | 21 mm x 19 mm |

Colors

blue or brown



Symblepharon shell by Dr Illig

The prostheses are used as a placeholder or as interim prostheses. A need for this exists e.g. after a plastic conjunctival surgery, plastic surgery of the eyelids including an involvement of the conjunctiva, in chemical burns or burns of the outer eye. The prostheses may also find use after enucleation. They are used to prevent and delay shrinkage and/or scarring of the conjunctival sac. Shape and size must be decided individually by the doctor.

Available sizes

- | | |
|---------|---------------|
| Size 01 | 25 mm x 28 mm |
| Size 02 | 24 mm x 27 mm |
| Size 03 | 22 mm x 26 mm |
| Size 04 | 21 mm x 25 mm |
| Size 05 | 20 mm x 24 mm |
| Size 06 | 19 mm x 23 mm |
| Size 07 | 18 mm x 22 mm |
| Size 08 | 17 mm x 21 mm |
| Size 09 | 16 mm x 20 mm |
| Size 10 | 15 mm x 19 mm |




Interim prostheses

The prostheses are used as a placeholder or interim prostheses. A need for this exists e.g. after enucleation and secondary orbital implants. It is used to prevent and delay shrinkage and/or scarring of the conjunctival sac. Shape and size must be decided individually by the doctor.

Interim prostheses white acrylic

Available sizes

- 18 mm x 22 mm
- 17 mm x 21 mm
- 16 mm x 20 mm

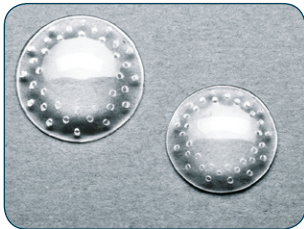

Contact shell by Prof. Walser

The contact sleeves are used afterwards keratoplastic surgery. Shape and size must be decided individually by the doctor.

Available sizes

- | | |
|---------|---------|
| Size 01 | Ø 25 mm |
| Size 02 | Ø 24 mm |
| Size 03 | Ø 23 mm |
| Size 04 | Ø 22 mm |
| Size 05 | Ø 21 mm |
| Size 06 | Ø 20 mm |